
i-BASEWAX

Modelling wax

Vaškas modeliavimui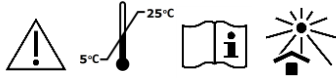

INSTRUCTION FOR USE

EN

INDICATIONS

- For the construction of the wax rim (for the record of central occlusion; for the selection of size and position of the anterior teeth)
- For the construction of the wax denture with set artificial teeth ("try in" patients' mouth; to test occlusal contacts)

DESCRIPTION

Modelling wax is a pink thermoplastic plate made of dental wax. Modelling wax can be easily softened over a flame, or otherwise, and modelled to the desired shape. Upon cooling, the wax retains its shape with great dimensional stability and excellent adhesiveness to the other dental materials involved.

INSTRUCTION FOR USE

Construction of the wax rim:

- Warm the wax plate and make a base on the patient's dental plaster cast model.
- Warm the other wax plate, twist the roller and press it on the top of the alveolar bone ridge of the cast.
- Adjust manufactured wax rim by hot wax knife to match the occlusion height.
- Place the manufactured wax rim into the patient's mouth, ask to clench the wax rollers placed in the central occlusion position.
- Set the patient's central occlusion height.
- On clenched wax roller, mark the guidelines for teeth size and position selection: anterior teeth and centre lines, position of canines, the smile and rest lines.

Construction of wax denture with artificial teeth set:

- Warm the wax plate, apply to the patient's dental plaster cast model.
- Model wax dental construction: set artificial teeth upright into wax base, shape artificial gums and any other base.
- Try the wax dental construction in the patient's mouth, test occlusal contacts.
- If wax construction fits well, finalize and prepare it for resin polymerization and pressing.

WARNINGS AND PRECAUTIONS!

NOTE! Do not use i-BASEWAX if patient is allergic to product or to any of the ingredients.

Prevention:

Wash hands thoroughly after handling. It is recommended to wear protective gloves/protective clothing/eye protection/face protection.

STORAGE

Keep tightly closed in dry well-ventilated place at 5-25°C. Protect from direct sunlight and heat sources. The batch number should be quoted in all correspondence. See packaging for batch and expiry date. Do not use after the expiry date. Dispose of contents/container to as required by national regulatory requirements. Keep out of the reach of children!

MANUFACTURERS RESPONSIBILITY

Our products have been developed for professional use in dentistry. As the application of our products is beyond our control, the user is fully responsible for the application. Of course, we guarantee the quality of our products in accordance with the applied standards.

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