i-BONDING® SE Universal

- DE** Lichthärtender selbstätzender Universalklebstoff
- EN** Light curing self-etching universal adhesive
- ES** Adhesivo Universal fotocurado de autograbado
- FR** Adhésif universel auto mordant à photopolymérisation
- LT** Šviesioje kietėjantis savaiminio ėsdinimo universalus surišėjas
- PL** Samowytawiający uniwersalny bond światłoutwardzalny
- RU** Светоотверждаемый самопротравливающий универсальный адгезив
- UA** Світлотверднучий самотравлюючий універсальний адгезив

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INSTRUCTION FOR USE EN

INDICATIONS

- To bond light or dual curing composites and compomers to tooth structure (dentin and enamel).
- To bond inlays, onlays, crowns, bridges, veneers in combination with a light or dual curing luting materials.
- To improve retention of a restoration.

DESCRIPTION

i-BONDING SE Universal is a light curing single component universal self-etching adhesive which incorporates aspects of etching and bonding in a single bottle.

INSTRUCTION FOR USE

NOTE! Do not use if primary package is broken. It is recommended to use cofferdam during application of the device.

1. Prepare cavity as always. Cleaning with oil-free prophylaxis paste, such as i-FASTE is recommended. For deep cavities use calcium hydroxide liner or glass ionomer-based cement, such as i-BAS.
2. The surface must be not contaminated prior to applying the adhesive.
3. Shake the bottle well before each use.
4. Delicate push on bottle and dispense few drops of adhesive into dispensing cup. Do not force bottle.
5. Using a fully saturated micro applicator tip, apply i-BONDING SE Universal on the prepared surface.
NOTE! Turn off or turn away the lamp from the working surface so the material does not harden.
6. Under light pressure rub the applied adhesive using the same micro applicator for 20 seconds.
7. Blow the surface with strong air force for 5 seconds to evaporate excess solvent. Use a vacuum aspirator to prevent the bonding liquid from scattering.
8. Curing with light for 10 seconds. Use any polymerization lamp with light intensity $\geq 700 \text{ mW/cm}^2$ and the wavelength range of 400-500nm (blue light).
9. Apply luting cement (indirect restorations), such as i-FIX Duo or light curing restorative material (direct restorations), such as i-LIGHT^N, i-XCITE LC^N, i-FLOW^N and finish restoration.

NOTE!

- a) After the desired amount of adhesive extruded, immediately close the bottle cap.
- b) When removing i-BONDING SE Universal from refrigerator after storage, leave the bottle at room temperature for some time (about 15 minutes) until warm-up.
- c) For non-cut enamel surfaces, pre-etching with phosphoric acid gel, such as i-GEL^N is always recommended.

WARNINGS AND PRECAUTIONS!

NOTE! i-BONDING SE Universal is a highly flammable liquid. It contains methacrylate monomers and ethyl alcohol. Do not use i-BONDING SE Universal if patient is allergic to device or to any of the ingredients. Do not use i-BONDING SE Universal on exposed pulp.

Hazard and precautionary statements:

Uncured device causes serious eye irritation, skin irritation, may cause an allergic skin reaction, harmful if swallowed. IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. If eye irritation persists: Get medical advice/attention. IF ON SKIN OR MUCOSA: Wash with plenty of water. If skin/mucosa irritation or rash occurs: Get medical advice/attention. Take off contaminated clothing and wash before reuse. IF SWALLOWED: Rinse mouth. Call a Poison Center or doctor/physician if you feel unwell.

Prevention:

Wash hands thoroughly after handling. It is recommended to wear protective gloves/protective clothing/eye protection/face protection (respectively) for doctor and patient. Do not eat, drink or smoke when using this product.

Interaction with other substances:

i-BONDING SE Universal should not be used with products containing eugenol and aluminium or ferric compounds-based haemostatics.

STORAGE

Storage container tightly closed in dry well-ventilated place at 2-8°C (preferable in a refrigerator). Protect from direct sunlight. Keep away from heat/sparks/open flames/hot surfaces. — No smoking. The batch number should be quoted in all correspondence. See packaging for batch and expiry date. Do not use after the expiry date. Dispose of contents/container to as required by national regulatory requirements. Keep out of the reach of children!

MANUFACTURERS RESPONSIBILITY

Our devices have been developed for professional use in dentistry. As the application of our devices is beyond our control, the user is fully responsible for the application.

Of course, we guarantee the quality of our devices in accordance with the applied standards.

PACKAGING

REF IBSTP 5ml bottle, 5pcs micro applicators, cup

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