i-MPRESS® Putty

i-MPRESS® Light Body

i-MPRESS® Universal Activator

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EN

INDICATIONS

For making of detailed impressions for dental and orthodontic use.

i-MPRESS Putty:

- Initial impression material for two step, double mix, sandwich technique.
- Initial impression material for putty-wash impression technique.
- Impression material for study models, orthodontic models, keys.
- Protective isolation of teeth during fitting of prostheses.

i-MPRESS Light Body:

- Second impression material for two step, double mix, sandwich technique.
- Corrective material for putty-wash impression technique.
- Relining impression material.

i-MPRESS Universal Activator:

- Catalyst gel for i-MPRESS Putty and i-MPRESS Light Body.

DESCRIPTION

i-MPRESS Putty, i-MPRESS Light Body and i-MPRESS Universal Activator are condensation type silicone impression materials with perfect balance between rigidity and elasticity. May be used with all types of impression trays. Do not require special treatment before use.

INSTRUCTION FOR USE

NOTE! Do not use if primary package is broken.

1. Prepare customary or individual tray. Coating of a tray with a thin layer of adhesive is recommended. Follow instructions for use of adhesive.
2. Use hand protective gloves for hand mixing procedures.
NOTE! Do not use latex gloves.
3. Mixing ratio and procedures:
 - a. i-MPRESS Putty:
Using scoop remove i-MPRESS Putty paste from the container. Remove off any excess by passing a flat spatula over the top of the spoon. Spread dosed amount of putty paste on hand. Dispense i-MPRESS Universal Activator gel in length of 3 cm onto putty paste and knead thoroughly for 30 seconds until homogenous consistency and uniform colour shade is reached.
 - b. i-MPRESS Light Body:
Dispense i-MPRESS Light Body and i-MPRESS Universal Activator gel in equal lengths onto mixing pad. Mix both components for 30

seconds until homogenous consistency and uniform colour shade is reached.

NOTE! Close containers immediately after use.

4. Mixing time at 23°C: 30 seconds.
5. Working time at 23°C: 45 seconds.
6. Fill the impression tray immediately after mixing. Insert impression tray applying gentle pressure into the mouth. Allow the impression material to set in the mouth.
7. Setting time (time in the mouth) at 37°C: 3:00 minutes.
8. Check curing of the impression material intraorally prior removing from the mouth.
9. Take the impression out from the mouth in a single rapid movement. Ask the patient to rinse mouth.
10. Carefully rinse the impression with lukewarm running water to remove traces of saliva and blood. Drain off the excess of water.
NOTE! Avoid over drying.
11. Immediately spray with or immerse into the disinfecting solution (containing optionally: glutaraldehyde, hypochlorite, iodophor, dimethyl-didecyl-ammonium chloride, phenoxyethanol). Rinse the impression with cold running water. When using hydrogen peroxide as disinfectant, rinse thoroughly with lukewarm water to avoid bubbles forming. Disinfection procedure does not alter the potential of the impression for optimum performance.
12. Pour out the plaster model. Use of dental plaster such as i-STONE III (Type 3), i-STONE IV or i-STONE Extra (Type 4) is recommended.
13. If the plaster model cannot be immediately poured, drain off the excess of water, disinfect and keep it in the hermetically sealed plastic bag at 23°C.
NOTE! Do not leave the impression in open air or immersed in water. Avoid heat and direct exposure of sunlight. Time lapse permitted between removal of the impression from the mouth and commencement of pouring the gypsum model is up to 30 minutes to 72 hours.

NOTE!

- a) To ensure good adhesion between the initial and second impressions, the initial impression must be carefully cleaned and dried prior to further use.
- b) Saliva, blood, skin secretions contaminated surfaces, latex gloves, eugenol, haemostatic substances interfere setting process.
- c) Do not combine with addition type silicone impression materials (A-silicone).
- d) The working / setting time can be affected significantly by ambient temperature variations, variations in the speed and friction involved in mixing and mixing ratio:
increased ambient temperature, intensive mixing or kneading or overdosage of the i-MPRESS Universal Activator gel speeds up the working / setting time. Vice versa decreased temperature, not intensive mixing or kneading and underdosing of the i-MPRESS Universal Activator gel prolongs the working / setting time.
- e) To extend the processing time place the material in the refrigerator prior to use.
- f) Silicone oil separated during storage of i-MPRESS Light Body can be mixed. It has no negative effect on the working process or setting time.

WARNINGS AND PRECAUTIONS!

NOTE! The i-MPRESS Universal Activator contains tetraethyl salicylate. Do not use materials if patient is allergic to devices or to any of the ingredients.

Hazard and precautionary statements:

i-MPRESS Universal Activator is flammable material, causes serious eye irritation, harmful if inhaled, and may cause respiratory irritation.

IF ON SKIN (or hair): Take off immediately all contaminated clothing and wash it before reuse. Rinse skin with water/ shower. If skin irritation occurs: Get medical advice/ attention. **IF IN EYES:** Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. If eye irritation persists: Get medical advice/attention. **IF INHALED:** Remove person to fresh air and keep comfortable for breathing. Call a POISON CENTER/doctor if you feel unwell.

Prevention:

Keep away from heat, hot surfaces, sparks, open flames and other ignition sources. No smoking. Use only non-sparking tools. Avoid breathing vapours. Use only in well ventilated area. Wash hands thoroughly after handling. It is recommended to wear protective gloves/protective clothing/eye protection/face protection for doctor and patient.

STORAGE

Store in a tightly closed container in dry well-ventilated place at 4-25°C (i-MPRESS Putty, i-MPRESS Light Body) / 15-25°C (i-MPRESS Universal Activator). Protect from moisture, direct sunlight and heat sources, sparks, open flames and other ignition sources. Always close the package immediately after use of the material. Do not use after the expiry date. Dispose of contents/container to as required by national regulatory requirements. Keep out of the reach of children!

MANUFACTURERS RESPONSIBILITY

Our devices have been developed for professional use in dentistry. As the application of our devices is beyond our control, the user is fully responsible for the application. Of course, we guarantee the quality of our devices in accordance with the applied standards.

PACKAGING

REF IMPP1	i-MPRESS Putty, 910ml paste, spoon
REF IMPP2	i-MPRESS Putty, 200ml paste, spoon
REF IMPL1	i-MPRESS Light Body, 140ml paste
REF IMPL2	i-MPRESS Light Body, 40ml paste
REF IMPC1	i-MPRESS Universal Activator, 30ml gel
REF IMPKC	i-MPRESS Mini Kit: 200ml i-MPRESS Putty, 40ml i-MPRESS Light Body, 30ml i-MPRESS Universal Activator, mixing pad, spoon
REF IMPK1	i-MPRESS Kit: 910ml i-MPRESS Putty, 140ml i-MPRESS Light Body, 2x30ml i-MPRESS Universal Activator, mixing pad, spoon

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