
i-ZOE^N

- EN** Resin modified zinc oxide eugenol cement
- LT** Derva modifikuotas cinko oksido eugenolio cementas
- PL** Cement cynkowo-eugenolowy modyfikowany żywica
- RU** Смолами модифицированный цемент оксида цинка с эвгенолом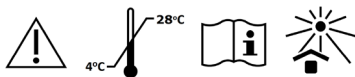

INSTRUCTION FOR USE EN

DESCRIPTION

i-ZOE^N is chemical curing, high strength, resin modified cement based on zinc oxide and eugenol for bases and linings under amalgams and glass ionomers; for temporary fillings; for temporary fixing crowns and bridges.

INSTRUCTION FOR USE

NOTE! Do not use if primary package is broken.

Dental fillings:

- Prepare surface as always. For deep cavities use calcium hydroxide liner.
- Mix powder and liquid just before application.
- Mixing ratio: 2 level spoons of powder / 3 drops of liquid.
NOTE! Mixing proportions variations may affect the strength and durability of the material.
- For mixing use metal spatula and glass slab. The powder should be added to the liquid in 2 portions (at least), until exact consistency is reached. Total mixing time is 60-90 seconds.
- The working area must be kept dry.
- Apply the mixed cement with suitable instrument into cavity. Total working time is 2-3 minutes including mixing.
- After 8 minutes from start of mixing restoration / cementation is ready for finishing.
- Higher amount of powder or higher temperature, higher humidity shorten the working / setting time and lower temperature, or higher amount of liquid prolong the working/setting time.

WARNINGS AND PRECAUTIONS!

NOTE! Product contains zinc oxide, poly(methyl methacrylate), silica, zinc acetate, eugenol. Do not use i-ZOE^N if patient is allergic to product or to any of the ingredients.

Hazard and precautionary statements:

Liquid contains eugenol, which may cause an allergic skin reaction, causes serious eye irritation, and may cause respiratory irritation IF ON SKIN: Wash with plenty of soap and water. Take off contaminated clothing and wash before reuse. If skin irritation occurs: Get medical advice/attention. IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. If eye irritation persists: Get medical advice/attention. IF INHALED: Remove victim to fresh air and keep at rest in a position comfortable for breathing. Call a POISON CENTRE or doctor/physician if you feel unwell.

Prevention:

Wash hands thoroughly after handling. Use only in well ventilated area. Avoid inhaling particles of powder dust. Wear protective gloves/protective clothing/eye protection/face protection.

Interaction with other substances:

Do not use i-ZOE^N with polymer based restorative materials.

STORAGE

Keep container tightly closed in dry well-ventilated place at 4-28°C. Protect from direct sunlight and heat sources. The batch number should be quoted in all correspondence. See packaging for batch and expiry date. Do not use after the expiry date. Dispose of contents/container to as required by national regulatory requirements. Keep out of the reach of children!

MANUFACTURERS RESPONSIBILITY

Our products have been developed for professional use in dentistry. As the application of our products is beyond our control, the user is fully responsible for the application. Of course, we guarantee the quality of our products in accordance with the applied standards.

PACKAGING

- REF ICEKP** 20g powder, 6g liquid, measuring scoop, mixing pad
- REF ICETP** 2x20g powder, 12g liquid, measuring scoop, mixing pad

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