# i-GEL<sup>N</sup>

- EN** Phosphoric acid etching liquid
- LT** Fosforo rūgštisėsdinimo skystis
- PL** Kwasa fosforowy w plynie
- RU** Жидкость для травления полостей зуба

## INSTRUCTION FOR USE EN

### INDICATIONS

– Tooth enamel and / or dentin etching.

### DESCRIPTION

i-GEL<sup>N</sup> is 37% thixotropic phosphoric acid liquid with optimum level of viscosity.

### INSTRUCTION FOR USE

**NOTE!** Do not use if primary package is broken.

It is recommended to use cofferdam during application of the product.

1. Prepare surface as always. The working area must be kept dry.
2. If the surface to be etched is not freshly exposed, cleaning with oil-free prophylaxis paste such as i-FASTE is recommended.
3. Apply layer of etch on all surface to be etched.
4. Leave etch liquid in place for 15 seconds (dentine), 30 seconds (enamel).
5. Rinse with water and dry with air.
6. Verify etched surface is clean and dry.
7. Use adhesive such as i-BONDING LC<sup>N</sup>.

### WARNINGS AND PRECAUTIONS!

**NOTE!** Do not use i-GEL<sup>N</sup> if patient is allergic to product or to any of the ingredients.

#### Hazard and precautionary statements:

i-GEL<sup>N</sup> contains phosphoric acid, which causes skin corrosion, severe skin burns and eye damage. IF ON SKIN (or hair): Remove/Take off immediately all contaminated clothing. Rinse skin with water/shower. Wash contaminated clothing before reuse. IF SWALLOWED: Rinse mouth. Do NOT induce vomiting. Subsequently, have the casualty drink 1 - 2 glasses of water immediately. Do not try to neutralize with bases, no charcoal. IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. Immediately call a POISON CENTER or doctor/physician.

#### Prevention:

Wash hands thoroughly after handling. Wear protective gloves / protective clothing / eye protection / face protection.

#### STORAGE

Keep tightly closed in dry well-ventilated place at 4-28°C. Protect from direct sunlight and heat sources. Do not use after the expiry date. The batch number should be quoted in all correspondence. See packaging for batch and expiry date. Dispose of contents/container to as required by national regulatory requirements. Keep out of the reach of children!

### MANUFACTURERS RESPONSIBILITY

Our products have been developed for professional use in dentistry. As the application of our products is beyond our control, the user is fully responsible for the application. Of course, we guarantee the quality of our products in accordance with the applied standards.

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