i-LIGHT[®]

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i-LIGHTTM is a light curing, radiopaque nano hybrid composite in Vita* shades based on UDMA, EOBDPADMA, BisGMA, TEGDMA resins and Zirconium-containing Silica and Barium glass filler with as follows average particle size distribution: main glass cluster size 0.7-1.5µm, nano glass cluster size 180nm. The inorganic filler loading is about 77% by weight. i-LIGHTTM is a high aesthetic material with excellent polishability, colour stability, high strength and abrasion resistance.

INSTRUCTION FOR USE

NOTE! Do not use if primary package is broken. It is recommended to use cofferdam during application of the product.

Preparation:

1. Prepare cavity as always. Cleaning teeth with oil-free prophylaxis paste, such as i-FASTE is recommended.
2. Use the minimally invasively technique for preparation of the cavity. All enamel margins in the anterior region must be bevelled. Do not bevel the margins in the posterior region and avoid slice preparations. Rinse with water and dry with air.
3. For deep cavities use calcium hydroxide liners, such as i-CAL Duo, i-CAL LC or glass ionomer base cement, such as i-BAS.
4. When filling cavities with approximal sections place a transparent matrix and fix it in place.
5. Apply layer of etch, such as i-GEL^N on all surface to be etched. Leave etch in place for 15 seconds (dentine), 30 seconds (enamel). Rinse with water and dry with air. Avoid over drying dentin. After drying, the surface must be chalky and do not contaminated prior to applying the adhesive. If the surface becomes contaminated with saliva, rinse and dry again and re-etch if necessary.
6. Apply a thin layer of adhesive, such as i-BONDING LC^N immediately onto etched surface and follow direction for use for adhesive.
7. The working area must be kept dry.

Restoration:

1. Apply the layer of composite with suitable instrument into cavity. Do not apply layers more than 2mm deep.
2. Light cure for 20 seconds per layer. Use any polymerization lamp in full mode (not ramp or pulse mode).
3. Due to the effect of the oxygen in the air, a thin smear layer of unpolymerized material remains on the surface of each layer. This bonds the layers chemically and must not be touched or contaminated with moisture.
4. Finish restoration.

Indirect method:

1. Use the minimally invasively technique for preparation of the cavity. To prevent the material fracturing, it must be at least 1.5mm thick in the lateral and vertical aspects. All internal line and point angles must be rounded. Avoid slice preparations. Prepare a flat cervical shoulder - do not bevel it. Any unavoidable undercuts must be blocked out with glass ionomer cement as i-BAS. Use slightly tapering diamonds with rounded tips for the preparation. For deep cavities use calcium hydroxide liners, such as i-CAL Duo, i-CAL LC.
2. Take the impression and fabricate temporary restoration, using acrylic resin such as i-TEMP. For cementation of temporary restoration use only non-eugenol cements, such as i-FIX.
3. Cast the impression with dental stone, such as i-STONE II, III, IV in the laboratory. Allow the model to set and pull off the impression. Block out the undercuts and apply an oil-free separating agent to the model. Build up the inlay layers on the model. Build up the approximal and deep occlusal sections first. Do not apply layers more than 2mm deep. Light cure for 20 seconds per layer. The finished inlay is then released from the die and cured fully for 8 minutes in curing oven. Trim the occlusal surface with fissure burs and polish to a high lustre with silicone polishers and diamond paste. Clean the inlay with liquid soap and water, rinse and dry with air.
4. Remove the temporary restoration and clean the cavity. Exert gentle pressure on the inlay to check for precision of fit. Do not use force. If necessary, trim the fitting surfaces to improve the fit.
5. The occlusion and articulation must not be checked when trying the inlay in as this would risk fracturing it.
6. Apply layer of etch, such as i-GEL^N on all surface to be etched. Leave etch in place for 15 seconds (dentine), 30 seconds (enamel). Rinse with water and dry with air. Avoid over drying dentin. After drying, the surface must be chalky and must not be contaminated prior to applying the adhesive.
7. Apply a thin layer of adhesive, such as i-BONDING LC^N immediately onto etched surface and follow direction for use for adhesive.
8. For cementation use any available dual curing luting composite. Apply the mixed material with suitable instrument on working area. Carefully press the inlay into position and remove excess of the material. To make certain that the inlay does not slip out of its proper position, press on it gently with a ball-end plugger until it has been light cured fully. Light curing for 20 seconds from each side. Remove the excess material with fine-grid diamonds and diamond finishing strips. Check the occlusion and adjust if necessary. Finish and polish.

NOTE!

- a) After the desired amount of material extruded, screw syringe plunger anticlockwise by a half to full turn to release residual

pressure inside the syringe and immediately close the syringe cap, so that the material is not unlighted.

- b) The material is sensitive to light. Avoid too long manipulation time under intensive lighting!

WARNINGS AND PRECAUTIONS!

NOTE! The composite contains methacrylates. Uncured methacrylates may cause allergic reactions. Do not use i-LIGHTTM if patient is allergic to product or to any of the ingredients.

Hazard and precautionary statements:

Product causes skin, serious eye irritation, may cause respiratory irritation, and may cause an allergic skin reaction. IF ON SKIN: Wash with plenty of soap and water. If skin irritation occurs: Get medical advice/attention. Take off contaminated clothing and wash before reuse. IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. If eye irritation persists: Get medical advice/attention. IF INHALED: Remove victim to fresh air and keep at rest in a position comfortable for breathing. Call a POISON CENTRE or doctor/physician if you feel unwell.

Prevention:

Avoid breathing vapours. Wash hands thoroughly after handling. Use only in a well-ventilated area. Wear protective gloves/protective clothing/eye protection/face protection.

Interaction with other substances:

i-LIGHT^N should not be used with products containing eugenol.

STORAGE

Keep container tightly closed in dry well-ventilated place at 4-28°C. Protect from direct sunlight and heat sources. Do not freeze the product. Do not use after the expiry date. Dispose of contents/container to as required by national regulatory requirements. Keep out of the reach of children!

MANUFACTURERS RESPONSIBILITY

Our products have been developed for professional use in dentistry. As the application of our products is beyond our control, the user is fully responsible for the application. Of course, we guarantee the quality of our products in accordance with the applied standards.

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* Registered trademark of the Vita Zahnfabrik H.Rauter GmbH & Co. KG, Bad Säckingen, Germany.