
i-SOL

- EN** **Conditioner for glass ionomer cements**
- ES** **Acondicionador para cementos de ionómero de vidrio**
- LT** **Kondicionierius stiklo jonomeriniams cementams**
- PL** **Kondycjoner do cementów glasjonomerowych**
- RU** **Кондиционер для стеклоиономерного цемента**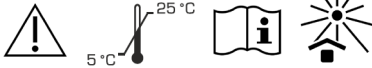

INSTRUCTION FOR USE EN

DESCRIPTION

i-SOL is a chemical curing conditioner to condition the surface of the tooth.

INSTRUCTION FOR USE

- NOTE! Do not use if primary package is broken.
1. Prepare surface as always. For deep cavities use calcium hydroxide liner.
 2. Apply i-SOL on prepared surface for 10 sec.
 3. Wash with water and dry with air.
 4. Apply glass ionomer cement, such as i-FIL, i-FIL LC, i-SIL and finish restoration.

WARNINGS AND PRECAUTIONS!

NOTE! Do not use i-SOL if patient is allergic to product or to any of the ingredients. **Hazard and precautionary statements:** Wear protective gloves/protective clothing/eye protection/face protection. Wash hands thoroughly after handling. Avoid contact with eyes. IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. Get medical advice/attention as soon as possible.

STORAGE

Keep tightly closed in dry well-ventilated place at 5-25°C. Protect from moisture, direct sunlight and heat sources. The batch number should be quoted in all correspondence. See packaging for batch and expiry date. Do not use after the expiry date. Dispose of contents/container to as required by national regulatory requirements. Keep out of the reach of children!

MANUFACTURERS RESPONSIBILITY

Our products have been developed for professional use in dentistry. As the application of our products is beyond our control, the user is fully responsible for the application. Of course, we guarantee the quality of our products in accordance with the applied standards.

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